

RJ Cooper & Assoc.
1-800-RJCooper
www.rjcooper.com infoRJ@rjcooper.com

Cordless Super-Switch (H-2)

The purpose of the Cordless Super-Switch is to function as a mouse-click for special software, or to allow other switches to do the same. 2 switch inputs along its side allow connection of other switch (es) to perform just about any action for any software (default is left/right click for Windows, left/control-click for Mac). Also, the Super-Switch functions as a regular switch for adapted toys and appliances!

If you're using other software than ours, it is very important that you confirm what it is actually *looking* for, when it refers to "switch." All our software looks for mouseclicks when we refer to a "switch." See Software Installation below if your software is looking for something else.

Hardware Installation for Windows


Plug the receiver into any available USB port, while your computer is on. Windows 2000/ME/XP/Vista/7 should detect it and install without any help from you. Windows 98 (Second Edition) will probably ask you to put in your Windows CD (the one that came with your computer). Do so and wait about 10 seconds for the CD to 'mount', then click OK.

Hardware Installation for Macintosh

You can plug the small receiver into any available USB port, either directly in the computer or in the keyboard, at any time.

Turning the Super-Switch On/Off (*CRITICAL*)

On the bottom of the Super-Switch is a hole. The very small white plastic part slides from far left to the middle, towards the dot on the right, for ON (you'll need to use a pen/pencil point to reach it; DO NOT JUST 'DIG AROUND' IN THERE! WE ARE NOT RESPONSIBLE FOR BROKEN ON/OFF SWITCHES. USE A FLASHLIGHT.)

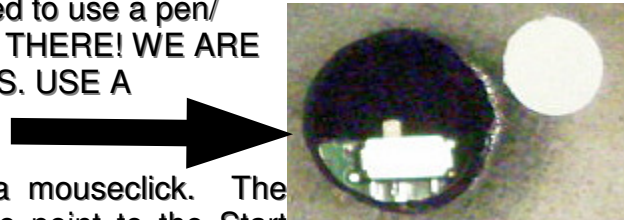

Now your Cordless Super-Switch should function as a mouseclick. The easiest way to test this is simply to use your mouse to point to the Start button (in Windows) or the Apple menu (on a Mac) and press the switch. The menu should appear/disappear with each switch activation.

Turn the Super-Switch OFF for toy/appliance use

Use the included cable to connect between switch input #1 (on the left) to your toy/appliance controller.

Software for Programs That Don't Look for a MouseClick

Remember, to control other company's special needs software, you need to know what that software is expecting as the "switch." Use our free SwitchHopper (Windows or Mac OS 9) or SAM-X (Mac OS X) software that is on our CD or our website, to set up the Cordless Super-Switch for those programs that are looking for a <space> or <control-enter> or <whatever> in their switch software. The Cordless Super-Switch can be set to 'send' any keystroke/key-combination or mouse button action to your computer.

For returns/repair, you must contact us and obtain an RMA from <techsupport@rjcooper.>